




# ACCU-CHEK® FlexLink

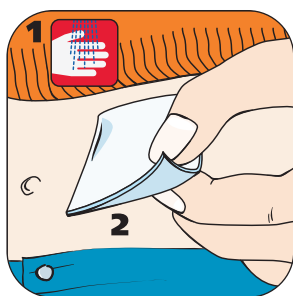


**Picture 1**  
1) Clean hands. 2) Choose a site away from the waistline, bones, scar tissue, belly button and recently used infusion sites. Allow the injection site to dry before inserting the Accu-Chek FlexLink.

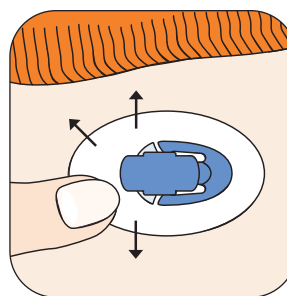


**Picture 5**  
Smooth out the self-adhesive and run your fingers over it. This ensures good skin contact.

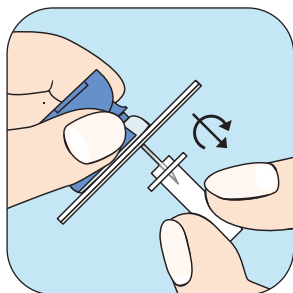


**Picture 2**  
Hold the Accu-Chek FlexLink at the blue holder cap with thumb and forefinger. Then turn and pull to remove the transparent protective cover.

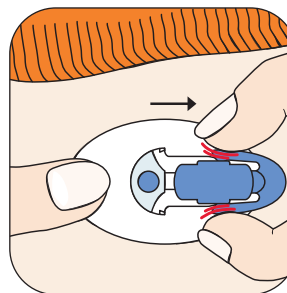


**Picture 6**  
Hold the self-adhesive in place, gently press the side clips of the blue holder cap and pull it from the cannula housing.

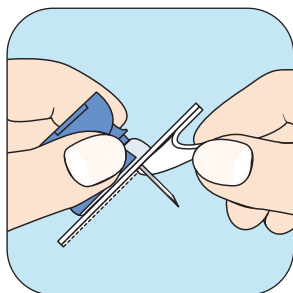


**Picture 3**  
Remove both pieces of the adhesive backing from the self-adhesive.

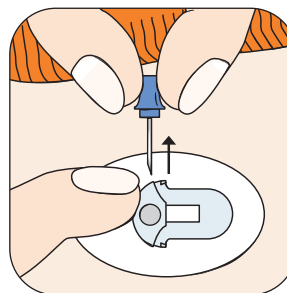

**Picture 7**  
To remove the introducer needle, hold the cannula housing firmly to your body. Using its handle, withdraw the introducer needle in a straight angle.

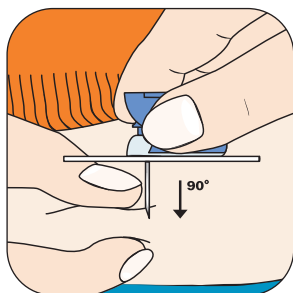

**Picture 4**  
Pinch the subcutaneous tissue. Insert the introducer needle and soft cannula quickly and straight into the infusion set, at a 90° angle.

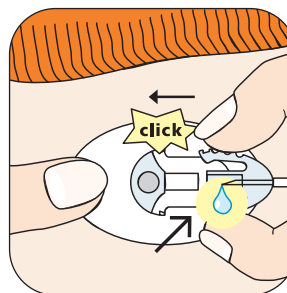

**Picture 8**  
Screw the luer-lock firmly into the screw thread of the pump adapter. Fill the Accu-Chek FlexLink according to the pump manufacturer's operating instructions until the medication emerges at the tip of the connector needle. Do not leave air bubbles in either the cartridge or the transfer set. Connect the transfer set to the newly inserted head set. A click must be heard when connecting. Fill the empty space of the cannula housing with a bolus of 1.0 units of insulin U-100. Start distribution with the pump.

Please refer to the Instructions for Use of the Accu-Chek FlexLink for detailed information.

ACCU-CHEK and LIVE LIFE. THE WAY YOU WANT., ACCU-CHEK FLEXLINK are trademarks of Roche.  
©2005 Roche Diagnostics



[www.accu-chek.com](http://www.accu-chek.com)  
Disetronic Medical Systems AG  
CH-3401 Burgdorf, Switzerland

**ACCU-CHEK®**

Live life. The way you want.