(EN) (DA) (NO) (SV) (FI) (EL) (RU)

Accu-CHEK® Mobile

05191165

(EN) Control

IN VITRO DIAGNOSTIC MEDICAL DEVICE. DO NOT INGEST!

Read this package insert, the package insert of the Accu-Chek Mobile test cassette and the User's Manual of the Accu-Chek Mobile blood glucose meter before using these control solutions. If you have any questions, contact your customer support and service centre.

This package insert features the following 2 symbols:

 This symbol indicates **a possible risk of injury or of damage to your health**.

 This symbol draws your attention to **important information**.

Intended use

- The control solution is intended for performing control tests on Accu-Chek Mobile blood glucose meters and Accu-Chek Mobile test cassettes.
- The control solution is suitable for self-testing.

Additional information

-  If the control test results are outside the specified concentration range, correct functioning of the meter and test cassette can no longer be assured. This may lead to blood glucose tests delivering incorrect test results. Incorrect test results can cause the wrong therapy recommendation to be made and thus produce serious adverse health effects.
- The applicators are for single use only.
- Keep the blood glucose monitoring system with all its components out of reach of children under 3 years. There is a risk of suffocation if small parts (e.g. covers, caps or similar objects) are swallowed.

-  The result of a control test must be within the concentration range displayed by the meter. The concentration range is also shown on the cardboard box of the test cassette. The concentration range specifies the permitted results for control tests and is determined for each production lot of test cassettes (batch of test cassettes).
- You can find information on the performance characteristics of the Accu-Chek Mobile system in the package insert of the Accu-Chek Mobile test cassette.
- All components of the pack can be discarded in domestic waste. Discard empty or partly used applicators according to local regulations.

Contents of the pack

- 2 applicators with control solution with a low glucose concentration (Control1, blue text on the label)
- 2 applicators with control solution with a high glucose concentration (Control2, red text on the label)
- 1 package insert

Regularly checking the meter

- You can check that the meter is delivering accurate test results. Always perform a control test using the Accu-Chek Mobile control solutions
- after cleaning the inside of the tip of the test cassette and the inside of the tip cover, as well as the measuring optics and the front opening of the meter,
 - if you are in doubt about a test result.

Additional materials required for performing a control test

- Accu-Chek Mobile blood glucose meter with User's Manual
- Accu-Chek Mobile test cassette with package insert

Storing and using control solutions properly

 Control solutions which are not properly stored or used can deliver results that are outside the specified concentration range.

- Use only control solutions which are within the use by date. The use by date is printed next to the  symbol on the bottom of the applicator and on the cardboard box.
- Store the control solution at temperatures between +2 and +30 °C.
- Control solutions that are stored outside this temperature range may become unusable before they reach their use by date.
- Control solutions must be at ambient temperature when you perform a control test. If you keep the applicators in a refrigerator, leave them to stand at ambient temperature until they have warmed up before you perform a control test.
- Do not leave the applicator open for a longer period of time. Apply the control solution to the test area immediately after opening the applicator.
- Do not re-moisten a dried or partially dry applicator brush. This may cause the control solution to deliver results that are too high.

Reagent composition

Composition in percent by weight

Control1: glucose (0.04 %), preservative (0.24 %) in phosphate buffer

Control2: glucose (0.12 %), preservative (0.24 %) in phosphate buffer

Last update

2019-05

Customer Support and Service Centre

United Kingdom

Roche Diabetes Care Limited

Charles Avenue, Burgess Hill

West Sussex, RH15 9RY, **United Kingdom**

Accu-Chek Customer Careline¹⁾

UK Freephone number: 0800 701 000

ROI Freephone number: 1 800 709 600

¹⁾calls may be recorded for training purposes

Some mobile operators may charge for calls to these numbers.

www.accu-chek.co.uk

www.accu-chek.ie

Australia

Roche Diabetes Care Australia Pty. Limited

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